

The clinical relevance of measuring NRBC in the XN-CBC

Sysmex Xtra Online | March 2012

The term ‘NRBC’ – ‘nucleated red blood cells’ – refers to precursor cells of the red blood cell lineage which still contain a nucleus; they are also known as erythroblasts or – obsolete – normoblasts. In healthy adults and older children, NRBC can only be found in blood-building bone marrow where they mature. Their appearance in peripheral blood points to extramedullary erythropoiesis (outside the bone marrow) or disruption of the blood – bone marrow barrier. Both possible scenarios can only be found in the course of a severe disease. Physiologically, NRBC occur in peripheral blood only in neonates and premature babies.

What does the NRBC count in neonates tell us?

It is a well-known necessity to determine the number of NRBC in blood samples from neonates in order to receive an accurate result of the white blood cell (WBC) count – either by manual correction or by an automated correction on the haematology analyser. Beyond that, counting the NRBC in peripheral blood from neonates can give valuable additional information. Already in the year 1924, reference values for the number of NRBC found in neonates on their first day of life have been published [1]. 41 of the 42 neonates examined in this study showed NRBC in peripheral blood, namely with an average number of 500 NRBC/ μ L or 0.1% of the circulating red blood cells, respectively. In the following decades, different authors showed similar results for neonates during their first days of life; some of these references are shown in table 1. The table also lists reference values for premature babies. In healthy neonates, the NRBC normally disappear from peripheral blood within the first week of life.

Reference (first author)	Sample number	NRBC/ μ L [$\bar{x} \pm 1$ SD]	Time after birth	Gestational age
Naeye[2]	84	919 \pm 1425	1 h	term
		560 \pm 771	6 h	term
Green[3]	102	400 \pm 1300	12–24 h	37–41 weeks
Green[4]	26	2900 \pm 3600	\leq 24 h	23–26 weeks
	37	1200 \pm 1800	-“-	27–29 weeks
	86	1000 \pm 900	-“-	30–32 weeks
Buonocore[5]	47	8521 \pm 1620	birth (cord blood)	24–27 weeks
	185	4548 \pm 473	-“-	28–36 weeks
	105	1689 \pm 290	-“-	37–41 weeks

Table 1 : Reference values from healthy neonates corresponding to time after birth and gestational age

Even in premature babies and neonates, knowing the exact number of NRBC allows drawing conclusions on their condition of health. Thus, NRBC counts greater than the indicated reference values may, for instance, refer to chronic or postnatal hypoxia, anaemia, maternal diabetes, or acute stress. Extremely increased values of up to 500 NRBC/100 WBC may even indicate an infectious disease such as congenital syphilis [6]. Although in 1–2% of the cases extremely increased NRBC concentrations might be idiopathic [7], they normally require rapid investigation of their cause in order to be able to start a potentially necessary treatment of the tiny patients as soon as possible.

Which relevance does the NRBC count have in older children and adults?

In adults and older children, the occurrence of NRBC in circulating blood always indicates a serious disease. NRBC may practically accompany almost every type of severe anaemia. Additionally, they are frequently seen in haemolytic anaemias, thalassaemias, haematological systemic diseases such as MDS or leukaemias, and severe bleeding, but they may also come along with a generally critical state of health (e.g. trauma patients from intensive care units). The overview article ‘Nucleated RBCs – Significance in the peripheral blood film’ by Constantino and Cogionis [8] gives a detailed listing of diseases associated with the occurrence of NRBC in peripheral blood and also explains the underlying mechanisms. As shown in the section *Studies on the clinical relevance of the NRBC count*, the occurrence of NRBC in the peripheral blood of adults also correlates with an increased mortality. Since this increases – among other factors – with the number of NRBC found, it is useful to determine their exact number for better risk assessment. Moreover, the NRBC concentration can indicate to which extent the bone marrow is stressed or diseased.

Methods for NRBC counting

Manual count

In the manual differential blood count, without a preceding analyser flag, any existing NRBC can be discovered only by chance. It is common practice in the microscopic examination to indicate the number of NRBC per 100 viewed WBC. Usually, 100–200 WBC (rarely up to 400 WBC) are viewed under the microscope. The problem of the manual count, however, lies in the fact that NRBC strongly resemble small lymphocytes; even experienced personnel can misclassify them as lymphocytes. This results in a falsely increased WBC number which may be particularly critical in leukopenic patients, for example during chemotherapy. Due to this fact and the high amount of work involved in the method, the manual count is unsuitable as a screening tool for NRBC. If the laboratory uses a haematology analyser which is at least able to generate a flag for NRBC, manual counting can take place somewhat more targeted. Nevertheless, the manual NRBC count has to be considered as critical for statistical reasons alone, with an average coefficient of variation of 40% (20–110%) [9].

Flow cytometric count

There are different possibilities of determining NRBC with the help of antibody-mediated flow cytometry. All of these have in common that an antibody sensitive against surface characteristics of the cells (CD; 'cluster of differentiation') is needed. Usually, an antibody against CD71 is used, which is only expressed on red blood cells (RBC), both mature and immature. To make sure that mature non-nucleated RBC can be differentiated from immature nucleated RBC, nuclei and DNA, respectively, are stained in addition. As an alternative, CD45, which is only expressed on white blood cells thus permitting a negative selection of the red blood cells, can be used instead of CD71. Although the flow cytometric method is quite accurate, its disadvantages prevail: by the employment of an antibody it is quite expensive, it is time-consuming, and it requires well-trained personnel, which makes it altogether not suitable for routine.

NRBC count on the Sysmex XN-Series

For more than 10 years, the counting of NRBC has been available on haematology analysers, but only with the introduction of the XN-Series it has become possible to determine NRBC routinely and accurately, without an increase in time, with each measurement. Since the NRBC count is included already in the complete blood count – the XN-CBC – waiting times for reflex tests and manual counts are eliminated and the potential errors of the latter avoided. Due to the auto-correction feature, WBC results are automatically corrected whenever necessary. Additionally, serious illnesses can be promptly recognised and therefore treated more rapidly and targeted. Patient monitoring is clearly facilitated under certain circumstances, which is particularly beneficial for patients e.g. from surgical wards where often merely a complete blood count is requested.

How does NRBC counting on the XN work? A special low-pH reagent stabilises the basophilic cells, which are counted in the same measuring channel, while all other WBC types are shrunk. The reagent lyses only the membrane of red blood cells and specifically labels the nucleus of any NRBC present. Interferences by lipids or lyse-resistant RBC are thus avoided. The exact mechanism is shown in fig. 1. Due to the resulting fluorescence and scattered light characteristics the cells can be separated as shown in fig. 2.

	Lysis		Label-ling		Fluorescence	Side scat-tered light	
Basophils		→		→		Strong	Strong
Lymphocytes		→		→		Medium	Medium
Monocytes		→		→			
Granulocytes (neutrophils, eosinophils)		→		→			
Nucleated red blood cells		→		→		Weak	Medium
Red blood cells		→		→		Very weak	Very weak

Fig. 1: Labelling mechanism in the WNR channel

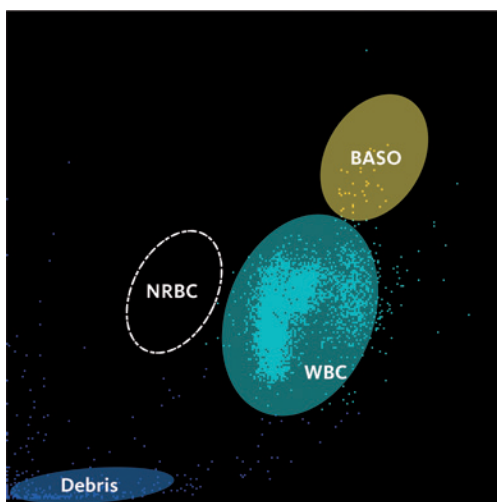


Fig. 2: WNR scattergram on the XN

With the haematology analysers of the XN-Series, NRBC can be accurately counted in both low and high measurement ranges and results are given as absolute (NRBC#) and relative values (NRBC%,/100 WBC) (fig. 3). Determining and displaying also the absolute count prevents e.g. the relative NRBC value to appear very low under the circumstances of a high absolute NRBC value and an extreme leukocytosis. The same applies for the reverse condition.

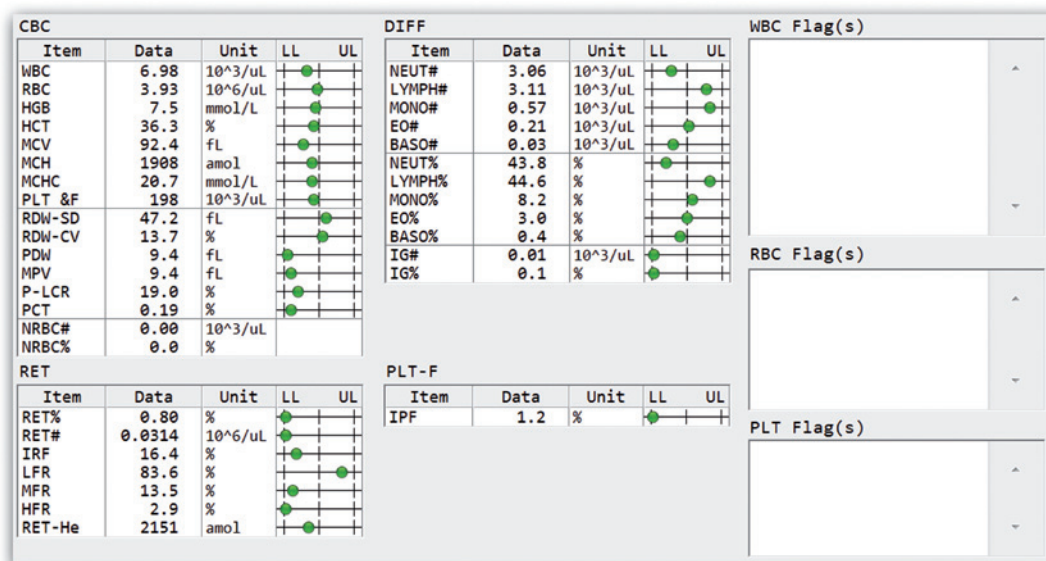


Fig. 3: Main screen view of a normal measurement result

Studies on the clinical relevance of the NRBC count

The below mentioned studies from the years 2002–2011 have all been carried out on the XE-2100 analyser. Nevertheless, the resulting conclusions can be applied to the XN-Series.

- Study of stem cell transplanted patients, performed at Jikei University Tokyo, Japan, published in 2005 [10]:
 - NRBC detection rates in the peripheral blood of 44 patients with different malignant haematological diseases were compared, starting from one month before stem cell transplantation until the time after engraftment of the transplant.
 - While directly after transplantation the detection rate and absolute NRBC counts increased equally in both survivors and non-survivors, the detection rate in the survivors group decreased after stem cell engraftment to 17%. In contrast, in the non-survivors group the detection rate increased even after stem cell engraftment to 100%.
 - Monitoring the NRBC count after stem cell transplantation could therefore provide useful clinical information on the severity of the disease and its progression.

- Study of thalassaemia patients, performed at two hospitals in Salerno and Palermo, Italy, published in 2009 [11]:
 - In this study, NRBC concentrations in the peripheral blood of 152 thalassaemia patients were evaluated and compared with the results of 26 patients with hereditary spherocytosis.
 - The number of NRBC reflected the ineffectiveness of the erythropoiesis: the highest numbers were found in thalassaemia syndromes with almost completely ineffective erythropoiesis whereas in patients with hereditary spherocytosis (effective erythropoiesis) no NRBC were detected.
 - Therefore, the NRBC count could be useful to determine and assess the ineffectiveness of erythropoiesis more exactly and potentially help to optimise transfusion therapy in patients with severe thalassaemia syndromes.
- Different studies of intensive care patients, performed at the Ruhr University Bochum, Germany, published between 2002 and 2011 [12–15]:
 - In one of the first studies of A. Stachon *et al.* [12] on this topic, NRBC values of 4,173 patients from the university clinic Bochum [general and accident surgery, heart surgery, neurology and internal medicine (without specialised haematology)] were evaluated during a period of 12 weeks. It was noticeable that the highest incidence of NRBC with 20% was seen in patients from the general and accident surgery and increased with age. Besides that, in this study population the mortality of NRBC-positive patients was clearly increased with 21.1% in comparison to NRBC-negative patients with 1.2%. Mortality increased with the number of NRBC found and the duration of their occurrence.
 - In following studies [13–15] the evaluators were able to deepen their investigations and confirm the results. It became apparent that the mortality of NRBC-positive patients was always higher than that from NRBC-negative patients (approximately 10-fold, depending on the particular study population). Altogether, the mortality of surgical intensive care patients with NRBC in their peripheral blood was the highest found.
 - Monitoring the NRBC count after stem cell transplantation could therefore provide useful clinical information on the severity of the disease and its progression.
 - An NRBC screening could help to promptly identify patients at high risk since the occurrence of NRBC in peripheral blood is a phenomenon appearing relatively early prior to death. In addition to that, NRBC-positive patients should receive longer intensive care treatment, because the mortality was clearly higher in patients positive for NRBC on the day of transfer to a normal ward than in those negative for NRBC on that day.

References

- [1] Lippman HS. (1924): Morphologic and quantitative study of blood corpuscles in the newborn period. *Am J Dis Child* 27 : 473–515.
- [2] Naye RL, Localio AR. (1995): Determining the time before birth when ischemia and hypoxemia initiated cerebral palsy. *Obstet Gynecol* 86 : 713–19.
- [3] Green DW. (1990): Mimouni G. Nucleated erythrocytes in healthy infants and in infants of diabetic mothers. *J Pediatr* 166 : 129–31.
- [4] Green DW, Hendon B, Mimouni FB. (1995): Nucleated erythrocytes and intraventricular hemorrhage in preterm neonates. *Pediatrics* 96 : 475–8.
- [5] Buonocore G, Perrone S, Gioia D, Gatti MG, Massafra C, Agosta R, Bracci R. (1999): Nucleated red blood cell count at birth as an index of perinatal brain damage. *Am J Obstet Gynecol* 181 : 1500–5.
- [7] Hanlon-Lundberg KM, Kirby RS, Gandhi S, Broekhuizen FF. (1997): Nucleated red blood cells in cord blood of singleton term neonates. *Am J Obstet Gynecol* 176 : 1149–56.
- [8] Constantino BT, Cogionis B. (2000): Nucleated RBCs – Significance in the peripheral blood film. *Lab Med* 31 : 223–229.
- [9] Tsuji T, Sakata T, Hamaguchi Y, Wang F, Houwen B. (1999): New rapid flow cytometric method for the enumeration of nucleated red blood cells. *Cytometry* 37 : 291–301.
- [10] Otsubo H, Kaito K, Asai O, Usui N, Kobayashi M, Hoshi Y. (2005): Persistent nucleated red blood cells in peripheral blood is a poor prognostic factor in patients undergoing stem cell transplantation. *Clin Lab Haem* 27 : 242–246.
- [11] Danise P, Amendola G, Di Concilio R, Cillari E, Gioia M, Di Palma A, Avino D, Rigano P, Maggio A. (2009): Nucleated red blood cells and soluble transferrin receptor in thalassemia syndromes: relationship with global and ineffective erythropoiesis. *Clin Chem Lab Med* 47 : 1539–42.
- [12] Stachon A, Sondermann N, Imohl M, Krieg M. (2002): Nucleated red blood cells indicate high risk of in-hospital mortality. *J Lab Clin Med* 140 : 407–12.
- [13] Stachon A, Kempf R, Holland-Letz T, Friese J, Becker A, Krieg M. (2006): Daily monitoring of nucleated red blood cells in the blood of surgical intensive care patients. *Clin Chim Acta* 366 : 329–335.
- [14] Stachon A, Segbers E, Holland-Letz T, Kempf R, Hering S, Krieg M. (2007): Nucleated red blood cells in the blood of medical intensive care patients indicate increased mortality risk: a prospective cohort study. *Critical Care* 11 : R62.
- [15] Kuert S, Holland-Letz T, Friese J, Stachon A. (2011): Association of nucleated red blood cells in the blood and arterial oxygen partial tension. *Clin Chem Lab Med* 49 : 257–263.
- [16] Hermansen MC. (2001): Nucleated red blood cells in the fetus and newborn. *Arch Dis Child Fetal Neonatal Ed* 84 : F211–F215.

